

Maintenance–Patient

Clean and Care for Tubing and Cannula

Provide your patient instructions on cleaning, disinfection and/or replacement information for the tubing and cannula.

3. Use a damp (not soaking wet) cloth or sponge.
4. Spray or wet the cloth or sponge with the mild detergent solution. DO NOT spray the cabinet.
5. Wipe down the cabinet.
6. To disinfect the Companion 5, use Lysol® Brand II disinfectant (or equivalent). Spray or wet a cloth or sponge with the disinfectant. DO NOT spray the cabinet or the LED/LCD display. Proceed as directed by the manufacturer.

Clean the Cabinet

To clean the cabinet do the following:

1. Turn OFF the Companion 5 and disconnect from AC power before any cleaning or disinfection activity.
2. Use mild detergent and water solution.

Patient Training Checklist

Use the following checklist as a guide to assist in setup and training a patient on the use of the Companion 5 and its accessories.

Patient Name:	
Companion 5 Serial #:	
Training Topic	Initials
Pre-Delivery Check List	
Indications for Use	
Contraindications	
Basic Concept Training	
Advise to read the Users Manual	
Safety Guidelines and Operational Safety Warnings/Cautions	
Locating the Companion 5	
Indicators	
Alerts and Alarms	
Companion 5 Maintenance	
Clean and Care for the Cannula per manufacturer's instructions.	
Clean the Cabinet as needed.	
Schedule Intake Filter Replacement Every 2 Years	
Trained By:	Date:

