

SET UP CHECKLIST FOR
 **GOLDEN**
LIFT CHAIR

_____ Inspect the chair with the customer/end user and note any damage or defects.

_____ Verify with the customer/end user that the chair is the correct model, size and color.

_____ Chair set-up

- Position the chair the correct distance from the wall.
- Plug in the chair and confirm there is power from the outlet.
- Adjust leg levelers as needed to make sure the chair is level.
- Recline the chair as far back as the particular model allows, making sure it does not touch the wall.

_____ Ride the chair through its entire range of motion, testing any accessory, (heat/massage, AutoFlex, etc.) to make sure it is functioning properly.

_____ Explain the SmartTek diagnostic system.

_____ Review the battery backup system. Install batteries if available. Note: Batteries should be replaced yearly, and **MUST** be replaced if the backup system is used.

_____ Review the proper use of the lift chair.

- Instruct on proper use of the hand control
- Review any accessories and how to use them.
- Demonstrate the proper way to get in and out of the lift chair.
- Review the proper position for the chair when not in use – seated.
- Review maximum weight capacity.
- Show them the owner's manual and review warranty information.

SAFETY PRECAUTIONS

- Do not allow children or pets near the chair while operating.
- Do not allow children to play with or operate the chair at any time.
- Your chair is made and designed for single person use and should be sized and fitted, including weight limit, accordingly

DEALER REPRESENTATIVE

CUSTOMER

DATE