



Maxi Sync Procedure

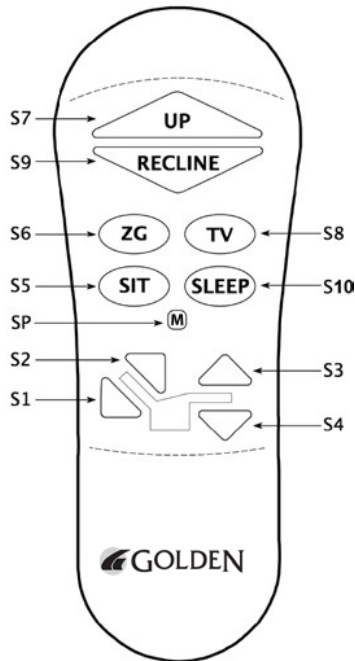

AutoDrive

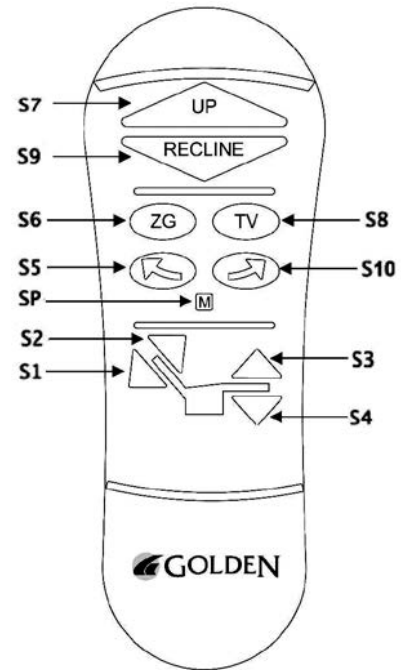

AutoDrive with Cradle

NOTICE The lift chair's electrical system is equipped with memory functions. Whenever any components (motor, control box, hand control, or cables) are replaced, the sequence below **MUST** be performed. Chair must be in standard sit position, and buttons pressed simultaneously until chair stops.

<p>Auto Drive Sequence:</p> <ol style="list-style-type: none"> 1. Press and hold S4 until the motor stops. 2. Press and hold S2 until the motor stops. 	<p>Auto Drive with Cradle Sequence:</p> <ol style="list-style-type: none"> 1. Press and hold S4 until the motor stops. 2. Press and hold S2 until the motor stops. 3. Press and hold S5 until the motor stops.
---	--

