



# Maxi Sync Procedure

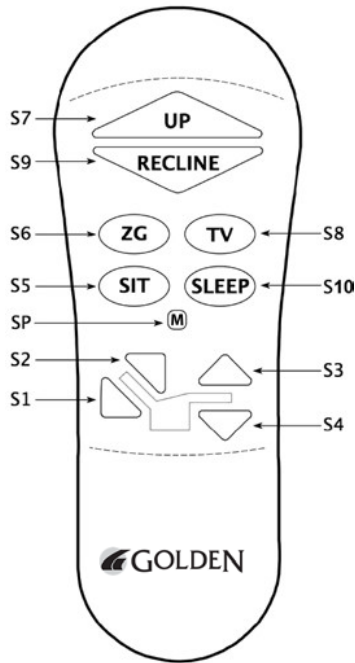

AutoDrive

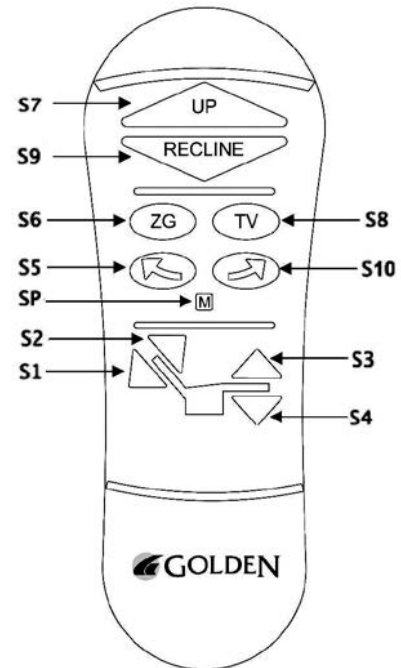

AutoDrive with Cradle

**NOTICE** The lift chair's electrical system is equipped with memory functions. Whenever any components (motor, control box, hand control, or cables) are replaced, the sequence below MUST be performed.

<p><b>Auto Drive Sequence:</b></p> <ol style="list-style-type: none"> <li>1. Press and hold S4 until the motor stops.</li> <li>2. Press and hold S2 until the motor stops.</li> </ol>	<p><b>Auto Drive with Cradle Sequence:</b></p> <ol style="list-style-type: none"> <li>1. Press and hold S4 until the motor stops.</li> <li>2. Press and hold S2 until the motor stops.</li> <li>3. Press and hold S5 until the motor stops.</li> </ol>
---	--

